


# SIMPLE STEPS TO IDENTIFY AND POWER CYCLE YOUR AED



## Locate the Serial Number

Remove your AED from its case and flip it over. Locate the Serial Number on the left side of the AED labeled with S/N. This number is required for the form completion below.

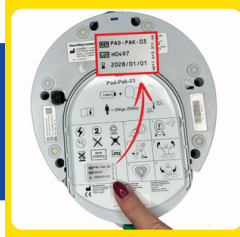

## Check the Expiry Date on Pad / Battery Pak

Always check the expiry date of pads and batteries in the HeartSine unit, found in a single cartridge with an hourglass symbol on the back of the AED, in the format (YYYY-MM-DD).



## Install the PadPak

When installing the Pad/Battery pak for the first time, slide it into the AED until you hear a "double click." For re-installation, pinch the tabs on the Pad Pak to remove and re-insert it into the AED unit.



## Check the Green Light

Verify that the green status indicator is blinking to indicate the initial self test routine has been formed and the device is ready for use.



## Press the POWER button to turn the AED on.

When turning on the AED, listen for instructions but DO NOT follow them to check for expected language prompts and no warning messages.



## Hear the Prompt

If you hear the message "Adult patient", or "Call for medical assistance" no further action is needed. If you DO NOT hear a prompt, please contact us directly.



## Turn OFF the device

To turn off the HeartSine samaritan PAD, press the On/Off button and ensure the flashing green status indicator indicates readiness for use.